WHEENO

FR	Sac pour fauteuil roulant Wheeno	2
EN	Wheeno wheelchair bag	4
DE	Der Wheeno Rollstuhl tasche	6
BG	Чанта за инвалидна количка Wheeno	8
HR	Wheeno torba za invalidska kolica	10
DK	Wheeno rullestolstaske	12
ES	Bolsa para silla de ruedas Wheeno	14
EE	Wheeno ratastooli kott	16
FI	Wheeno pyörätuolilaukku	18
GR	Τσάντα για αμαξίδιο Wheeno	20
HU	Wheeno kerekesszéktáska	22
IT	Borsa per sedia a rotelle Wheeno	24
IE	Mála do Chathaoireacha Rothaí Wheeno	26
LV	Wheeno ratiņkrēslu soma	28
LT	Wheeno vežimėlio krepšys	30
MT	Il-Borża Wheeno għall-Karru tal-Kumdità	32
NL	Wheeno rolstoel tas	34
PL	Torba na wózek inwalidzki Wheeno	36
PT	Bolsa para cadeira de rodas Wheeno	38
CZ	Taška pro invalidní vozík Wheeno	40
RO	Geanta pentru scaunul cu rotile Wheeno	42
SK	Taška pre invalidný vozík Wheeno	44
SI	Torba za invalidski voziček Wheeno	46
SE	Väska för rullstol Wheeno	48
JP	Wheeno車椅子用バッグ	50
KR	휠체어용 Wheeno 가방	52
CN	Wheeno轮椅背包	54

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Wheeno wheelchair bag

Product Description

The Wheeno bag is a transport accessory designed for wheelchair users. It is quick and easy to install thanks to an innovative Fidlock magnetic fastening system, offering practicality, security, and autonomy to its user.

Package Contents

- 1 Wheeno bag
- 2 magnetic mounting brackets (Fidlock system)
- 2 mounting screws

Compatibility

 **Important:** The Wheeno is compatible with wheelchairs that feature a horizontal (cross) rear bar, onto which the mounting brackets can be attached. This bar must be accessible and have a shape and diameter compatible with the brackets.

PRISM strives to expand its range of brackets to meet the needs of as many users as possible but cannot guarantee universal compatibility with all wheelchair models.

PRISM strongly recommends testing the compatibility of the bracket with your wheelchair before purchasing. Some resellers may offer in-store testing, and PRISM can provide guidance via email at contact@prism-offroad.com.

Assembly Instructions

Step 1: Attaching the Brackets

1. Clear the rear horizontal bar of your wheelchair of any elements that may obstruct the bracket installation.
2. Position the brackets on the bar, making sure to measure the exact spacing that corresponds to the attachments on the back of the bag. The centers of the two magnets must be exactly **15.5 cm** apart. Also, ensure they are centered on the bar to avoid imbalance between the right and left sides of the wheelchair.
3. Secure each bracket using the two provided screws. Use an appropriate screwdriver and tighten firmly to ensure a stable, long-lasting hold.
4. Over time and with use, the screws may need occasional re-tightening to maintain bracket stability. Feel free to check regularly, especially if the bracket seems loose.

Step 2: Installing the Bag

1. Bring the bag close to the brackets installed on the wheelchair: the back of the bag (which contains the female part of the attachment system) should face the brackets (which contain the male part on the chair). The male part should approach the female part from a slightly elevated angle.
2. The Fidlock magnets will automatically guide the bag into a secure lock.
3. To remove the bag, pull gently upward and then outward (away from the chair) to release it.

The Wheeno fastening system is designed to be simple and intuitive. However, attachment and removal may vary depending on each user's grip strength and range of motion. Some users may choose to turn slightly to grab the top handle, especially when the backrest of the chair is not too high. Others may prefer not to turn and instead reach behind the chair with both arms to grasp the side handles. Each user can adopt the method best suited to their mobility and comfort.

Care & Precautions

- Clean the bag with a damp cloth. Do not machine wash.
- Regularly check the tightness of the screws and retighten if necessary.
- Do not carry heavy objects that could unbalance the wheelchair or interfere with normal use.
- Do not expose the magnets to strong magnetic fields (MRIs, sensitive medical equipment, etc.).

Warnings & Liability Limitations

PRISM places great importance on the quality, safety, and accessibility of its products. However, it is the user's responsibility to ensure that the Wheeno bag is compatible with their wheelchair before installation.

By using this product:

- The user acknowledges that the bag may potentially hinder access or movement depending on the wheelchair configuration and individual needs.
- PRISM cannot be held liable for any discomfort, inconvenience, or incidents resulting from the use of the bag, improper installation of the brackets or bag, use on an incompatible wheelchair, or usage that deviates from the intended purpose.
- It is not recommended to carry fragile or valuable items in the bag. PRISM cannot be held responsible for damage or breakage of items stored in the bag or that may come into contact with it.

The use of the Wheeno bag is the sole responsibility of the user. In case of doubt or the need for specific adaptations, please feel free to contact our customer service to discuss possible solutions.

Legal & Safety Information

- This product is not a medical device.
- It has been designed in compliance with applicable safety standards for mobility accessories.
- Keep out of reach of children.
- Keep away from open flames, high heat sources, or any element that could ignite the fabric.
- Do not modify the fastening elements.
- The magnets used are strong; if you wear a pacemaker or have other medical implants, consult your doctor before use.

Warranty & Support

The Wheeno comes with a **2-year warranty** against manufacturing defects under the legal warranty of conformity. For any questions, support, or spare part requests, please contact our customer service at: contact@prism-offroad.com

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